

No - Contact Skin Surface Thermometer Infrared Skin Probe

THERMOMETER, NO-CONTACT INFRARED IR200 (40/CS)



1. Press Start/Scan button to ensure unit ready to use. Uses 2 AA Batteries
2. Remove hair and/or perspiration from forehead. Point thermometer towards forehead from a 2-inch distance
3. Press the Scan button. The Temperature is shown on the screen or a beep will sound in 1 second
4. Allow 3-5 second intervals between two measurements
5. Record Temperature per District Guidelines
6. Troubleshooting: See Manufacturer's Instructions